Easy•Touch® HealthPro™

Glucose Monitoring System

Quick Reference Guide

This guide is intended to be used as a quick reference to the EasyTouch® HealthPro™ User Manual. Please read the complete manual before using the EasyTouch® HealthPro™ Glucose Monitoring System.

Display Messages

Meter may be defective. Reboot meter with new batteries.

Test strip may be contaminated or used. Retest with new test strip.

Insufficient amount of sample blood placed on the test strip. Retest with new test strip.

Inadequate sample of blood or control solution on test strip. Retest with new test strip.

Blood sample is placed on the test strip before the symbol for blood blinks.

Color bars on the back of strip are dirty, bad or there is too much light. Retest with new test strip.

Ambient temperature is less than 50°F (10°C). Wait 10 minutes and retest with new test strip.

Ambient temperature is over 104°F (40°C). Wait 10 minutes and retest with new test strip.

Test result is lower than 20mg/dl. Repeat test to ensure accurate results.

Test result is higher than 600mg/dl. Repeat test to ensure accurate results.

Please refer to pages 38-40 in the [EasyTouch® HealthPro™ User Manual](#) for additional information.

Detailed instructions on setting up your meter with the date, time and number of days for test averaging can be found in the EasyTouch® HealthPro™ User Manual.

Performing a Test

STEP 1


Insert a test strip firmly into the test port. The meter will automatically turn on and read the test strip code. A blinking test strip “” will appear when the meter is ready for testing.

STEP 2

Place blood sample onto the end of the test strip. When the test strip has collected the blood sample (0.5µL), the meter will beep and automatically start counting down.

STEP 3

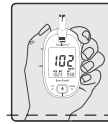After 5 seconds, your blood glucose result will appear on the display and will automatically be stored into the meter's memory. The meter will automatically turn off.

Reviewing Your Results

The EasyTouch® HealthPro™ Meter stores up to 365 test results in its built-in memory, along with the average sugar level for the number of days you preset the glucose test meter to calculate.


STEP 1

To retrieve your test results, press and release the “” button. Most recent results appears first. Press the “” and “” buttons to navigate through your previous test results.

STEP 2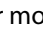

To retrieve your test averaging, press and release the “” button when viewing your most recent test result. To switch between day-averaging presets, press the “” and “” buttons.

For [Questions](#) or [Comments](#), call MHC Customer Support at (877) 358-4342, Monday thru Friday, 9am to 5pm EST.

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